Daily Aspirator Cleaner and Disinfectant

Date of revision: 17/01/25

Available in 5 Litre / 2.5 Litre / 1 Litre dispenser bottle

UK

Daily Aspirator Cleaner and Disinfectant

For the daily cleaning and maintenance of dental aspirator systems. Can be used on wet line, dry line and semi wet/dry line suction systems.

To Use Daily: Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes and then rinse thoroughly with clean water. For professional use only.


Efficacy (15 mins): Bactericidal. Yeasticidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN17111, EN14885, EN14562, EN14561, EN14476, EN13727, EN 13624.

Composition: Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Storage. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Aluminium. **Disposal.** Do not discharge concentrate into drains or rivers. P301+P330+P331: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P304+P340: IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Danger. H315: Causes skin irritation. H318: Causes serious eye damage. H410: Very toxic to aquatic life with long lasting effects. P280: Wear protective gloves, protective clothing, eye protection, face protection. P302+P352: IF ON SKIN: Wash with plenty of water. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. P332+P313: If skin irritation occurs: Get medical advice/attention. P391: Collect spillage.

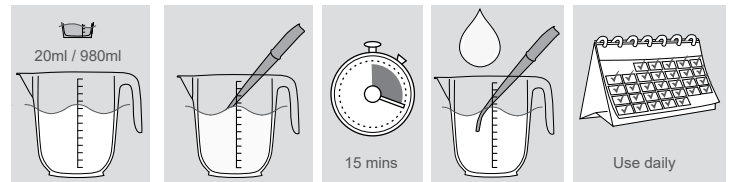Efficacy

Microorganism	Time
Bactericidal	15 min
Pseudomonas aeruginosa	15 min
Staphylococcus aureus	15 min
Enterococcus hirae	15 min
Yeasticidal	15 min
Candida albicans	15 min
Non enveloped viruses	15 min
Adenovirus 5	15 min
Murine norovirus	15 min
Enveloped viruses	15 min
Hepatitis B virus (HBV)	15 min
Hepatitis C virus (HCV)	15 min
HIV	15 min
Measles virus	15 min
Hepatitis Delta virus	15 min
Influenza virus	15 min
Coronavirus	15 min
Herpesviridae	15 min
Filoviridae (Ebola)	15 min
Rubella virus	15 min
Rabies virus	15 min
Poxviridae	15 min

UK	Danger.
BG	Опасно.
CS	Nebezpečí.
DA	Fare.
DE	Gefahr.
EL	Κίνδυνος.
ES	Peligro.
ET	Ettevaatust.
FI	Vaara.
FR	Danger.
HR	Opasnost.
HU	Veszély.
IT	Pericolo.
LT	Pavojus.
LV	Bīstami.
NL	Gevaar.
NO	Fare.
PL	Niebezpieczeństwo.
PT	Perigo.
RO	Pericol.
SK	Nebezpečnosť.
SL	Nevarno.
SV	Fara.

UK
CA
0120

Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk

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